

Simplist® MicroVault® Instructions for Use

Opening the MicroVault® Packaging

STEP 1

Inspect outer packaging.
Check expiration date.
Check that you have the right drug and the right dose.



STEP 2

Open at the perforated line.
Hold tube and cap upright at a 45-degree angle.
Twist to open until seal is broken.



STEP 3

Pull cap straight away from tube.




STEP 4

Visually inspect the syringe prior to administration.



What not to do when opening MicroVault® Packaging

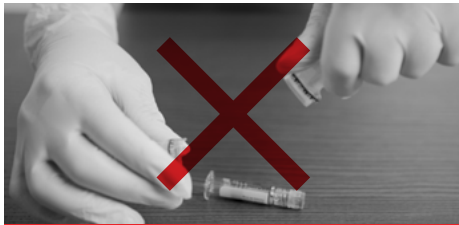

Do not open the MicroVault® at a downward angle as the syringe may fall out after twisting open.



Do not snap the MicroVault® cap



Do not put pressure on the MicroVault® cap

Instructions for Simplist® syringe use

STEP 1

Twist off the syringe tip cap.
Helpful hint: Before removing the cap, priming the syringe may be easier if you push the plunger rod forward with the tip cap on in order to break the seal.



STEP 2

Prime the syringe by expelling the air bubble.
Adjust the dose if applicable.



STEP 3

Administer dose. Do not leave syringe unattended. Discard syringe into appropriate receptacle.

Helpful hint: If clinically appropriate, according to the package insert, attach a needle per institutional guidelines.



Scan the QR code to
watch the training video
www.simplist-us.com/microvault-instructions-for-use/